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für unsere Abdruckschäume
Ideal · Genial XL · Diabetic Step · Maximal
Minimal · PodoScann · SlimLine · Kristall

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1. Preparation

note the name of the customer and date the impression was taken on the foam box. Open the box and with the Foam sides facing up at one. Lay a solid, level surface.

2. Heel

The patient's foot is centered on the foam sheet place and carefully on the Deposit foam surface. Well the check the perfect position of the foot. If the foot is not on the foam everywhere, correct now. Start with the By guiding your hand deep into the heel Press foam.

3. Metatarsus

Push the base M5 (midfoot) with the edge of the hand.

4. Big toe joint

Now press the big toe joint as well with the back of your hand in the foam. Relocate the power of your hand afterwards on your edge of your hand and use it to press the individual toes also.

5. Completion / repetition

Now take the foot vertically upwards from the foam. Repeat the the o.g. Processes with the other foot. Usability check the imprint, if necessary with a new one Repeat the pair of impression foams. Foam residues on the foot are made hygienic Reasons to remove immediately. FSG impression foam store separately with imprint.

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Instruction for use

1. Purpose

Skilled staff in orthopaedic technology needs FSG-Imprint-Foam to make footprints. Based on the diagnosis of an orthopaedic, the orthopaedic technician or orthopaedic shoemaker makes a negative mould of the foot. The orthopaedic technician can then process this negative mould with various techniques. Using various materials (e.g. plaster or casting resin), he can create a positive mould, or the footprint can help him determine prominent parts or the anatomy of the foot. It is also possible to scan the footprint and process it with specialized software. In most cases, an insole is manufactured, but there are also other options like e.g. the making of an individual made-to-measure shoe.

2. Storage and transport

FSG-Imprint-Foam has to be stored in a dry place. Ideal storage conditions are temperatures of 18° to 24° C and a relative humidity of 55 to 65%. Damaged and wet FSG-Imprint-Foam or FSG-Imprint-Foam boxes have to be sorted out and discarded. The disposal has to be carried out according to the legal and local requirements of the corresponding country. Please transport the FSG-Imprint-Foam with great care, i.e. do not throw, push or knock against it. The units must be stored standing, not lying! A maximum of 4 can be packed on top of each other.

3. Use

FSG-Imprint-Foam may be used by trained, specialized staff only. The user should wear gloves for his own protection. FSG-Imprint-Foam is suitable for external use and single use only. The FSG-Imprint-Foam must be dry and undamaged. The foot of the patient must be clean, free from cream and free from infections. If necessary, use a sterile film or bandage to cover the foot. Take the FSG-Imprint-Foam out of the box immediately before use. Avoid a contact of FSG-Imprint-Foam or its particles with mucous membrane. The FSG-Imprint-Foam must be unfolded and lie on a solid, flat surface. The foot must be guided by the user to guarantee a centred print. For a print with full weight (standing), make sure that the patient does not lose his footing. If necessary, support him. Check the usability of the print. If necessary, repeat the procedure with new foam. For reasons of hygiene, remove foam remains from the foot.

4. Shelf life

The FSG-Imprint-Foam can be stored up to maximum 36 months. Changes of colour do not impair the properties of the foam. The expiry date can be found on the label on the packaging unit. The year and month (XXXX-XX) of the product's maximum shelf life are shown behind the "hourglass" symbol.

5. Disposal

FSG-Imprint-Foam can be disposed of via usual domestic refuse, the cardboard box can be disposed of in a paper-recycling collection container. Contaminated FSG-Imprint-Foam has to be disposed of as appropriate for the kind of contamination in question.

Instruction for use in other languages:

www.formschaum-gmbh.de/en/mdgba